

Instructions for use: Nacera® Blue X

Application

⚠ General Note

Nacera® Blue X must be stored at temperatures 10°C to 40°C. The liquid must be protected against direct sunlight. For a contamination free use of Nacera® Blue X only use metal free brushes. Shake well before use!

⚠ Danger

Please use your personal safety equipment especially safety glasses during the handling of Nacera® Blue X. The liquid is not suitable for consumption. It must be disposed of properly. Further information is given in the safety data sheet.

i Safety Instructions

Please read the instructions carefully before using Nacera® Blue X. They contain important information necessary for correct processing and for the safety of the patients and users. The Nacera® Blue X liquid is produced and tested to the highest quality standards. To guarantee this quality also during processing, the protocol described here must be strictly followed. This ensures the high quality of the restorations.

↑ INDICATION

Indication

Nacera® Blue X is part of the Nacera® System and may therefore only be used in conjunction with products from the Nacera® Pearl series. The liquid increases the translucency in the incisal area where applied. The aim is to create an aesthetic and lifelike dental enamel.

↓ INDICATION

Contraindication

Nacera® Blue X is contraindicated in cases of bruxism. Do not treat bridges with more than three units, static heavily loaded crowns/constructions/areas like e.g. pontics basal side and connectors with Nacera® Blue X.

Nacera® Blue X is only allowed to be used with products of Nacera® Pearl. For further information read the instructions of Nacera® Pearl Zirconia.

MAX. 3 UNITS			ONLY OUTSIDE			NO CONNECTORS		
NO PONTIC			NO SUBSTRUCTURE			NO ABUTMENTS		

Application

Application

We recommend to apply **Nacera® Blue X**, for the desired result in the buccal / incisal / occlusal area of the crowns / bridges.

Shake well before use! To avoid any contamination only use a metal free brush.

Please don't work directly from the bottle, always use a separate tray and apply with a metal free brush dedicated only to the use of **Nacera® Blue X**. Please perform the following steps 1 to 4. E.g. on anterior crown:

1. **Nacera® Blue X** should be painted vestibular from the middle to the incisal.
2. **Nacera® Blue X** should be painted from the upper third to incisal.
3. **Nacera® Blue X** should be painted in the upper quarter to incisal.
4. Repeat step 3

The blue indicator colour gives you control over the amount of **Nacera® Blue X** applied.

Don't pour the used **Nacera® Blue X** back to the bottle.

Sintering

Please dry the crowns processed with **Nacera® Blue X** for 15 minutes before further processing.

Please sinter the crowns treated with **Nacera® Blue X** according to the sintering cycles given in the instructions of **Nacera® Pearl**.



If you prefer another language, please scan the QR-Code below or visit:
doceram-medical.com/en/downloads/



You prefer a video tutorial? Visit our YouTube-Channel!
www.youtube.com - Search "Nacera Inside" or scan the QR-Code.

Brushing Technique Anterior

START	0			1			2	
	3			4			5	
Important: Only use metal free brush! Shake well before use!								