



**CROWN FINISHED
IN 90 SECONDS**

NACERA® SHINE ZR

INSTRUCTIONS FOR USE

Nacera® Shine Zr is a diamond polishing paste to achieve a long-lasting high-quality shine on zirconium oxide. It contains a very high diamond content and an exceptionally hard consistency - for less work and a glossier finish.

Nacera Shine Zr can be used for pre-polishing with standard hard brushes or for high-gloss polishing with goat hair brushes, cotton or leather buffing wheels. Its exceptionally hard consistency and

a special wax/acid mixture that acts as a binding and adhesive agent ensure that the polishing tool only takes up the quantity required for the particular operation.

This avoids any spraying or smearing of excess material, which in turn results in greater efficiency and cleanliness during processing of zirconium oxide and reduced consumption of high quality diamond paste.

- ✓ High gloss finish of the fully-ceramic restoration in 90 seconds
- ✓ long-lasting high-quality shine on zirconium oxide.
- ✓ very high diamond content and an exceptionally hard consistency - for less work and a glossier finish
- ✓ easy to handle
- ✓ efficient in terms of consumption
- ✓ effective and time-saving to use



Not for intraoral use



pre-polishing:
5.000 - 10.000 RPM



high gloss finish:
5.000 - 15.000 RPM



You prefer a video tutorial? Visit our YouTube-Channel! www.youtube.com - Search "Nacera Inside" or scan the QR-Code.